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GB

- disinfectant and liquid soap dispenser
- with lever and disposable pump
- suitable for container shapes from all leading manufacturers

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GB

Disinfectant and liquid soap dispenser

Before installation:

Unpack the box and remove the plastic packaging of the dispenser. The following items are included:

- 1 dispenser
- 1 operating lever (to be installed at the end (see 1a)
- 1 wall plate for wall mounting (not necessary, if a suitable plate had been already installed)
- 1 drip tray
- 1 bag with a set of three screws and dowels
- 1 operating instructions in German, English and French

Installation / operation

1 a) completion of the dispenser

For packing reasons the dispenser is supplied with separated lever (item 1).

For mounting

- remove the wall plate (see 1b)
- remove the metal strip on the back of the dispenser; then put the lever and push it until it clicks into place. Put the metal strip back again; It serves to secure the lever and simultaneously it is the dispensers' upper point of attachment to the wall plate

1 b) Assembly of the dispenser

- open the dispenser by pressing the push button at the top (item 2) and lift the lid towards you
- press the locking tab through the cutout at the rear part of the dispenser, remove wall plate from the housing by pulling it downwards
- attach the wall plate with screws and dowels (item 3) (not necessary, if the wall panel already exists)
- push the dispenser from the top downwards the wall pannel, until the locking tab clicks into place

Putting into operation

Push the full container (disinfectant or soap) by pushing it upwards over the pump and put it into the housing (item 4). Close the lid and place the drip tray. On first use, move the lever several times downwards until it stops. Collect the liquid coming out of the tip of the pump in a suitable vessel.

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Note: Before installation determine the position of the dispenser carefully!

- the dispenser may be attached over a sink - however, as it has a drip tray, this is not necessary
- mounting height above the sink: 15-20 cm (sink - bottom of dispenser)
- mounting height on the wall: 120 cm (floor- bottom of dispenser)
- if you want to mount the dispenser in an hallway, install it high enough in order to prevent damages by service carts or similar things

3 Dosage

At delivery the pump is set at the maximum dosage of 1,8 ml per stroke. Dosage can be reduced to 1,2 ml or 0,8 ml per stroke by moving the switch to the left or right (item 5).

4 Cleaning of dispenser / pump replacement

Basically the dispenser is maintenance free. For reasons of hygiene and infection prevention, we recommend to clean and disinfect regularly.

Clean the housing only with mild detergents. If desired, treat additionally with surface a cleaner or a disinfectant. The housing can also be washed in a dishwasher.

In order to prevent any hygiene problems, we do not recommend a cleaning of the pump.

The plastic pump is disposable and may be replaced economically (possibly cheaper than a dismanteling and cleaning).

If considered to be necessary, remove pump and dispose it properly.

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